

REQUIRED FORMS

- Proof of Insurance (photocopy of insurance card)
- Patient Demographics (photocopy of Face sheet)
- Two most recent follow-up visit notes (photocopies of notes)
- Requisition form
Correct and complete specimen identification is essential for accurate test results.

CRITERIA FOR SPECIMEN REJECTION

- There are less than 2 patient identifiers on tube
- There is less than 7 mL (7 cc) sample volume
- The sample receipt date is over 5 days from date of collection
- Gestational age less than 10 weeks

Contact Premier Genomics at 1-888-416-8118 with questions about sample preparation and shipping.

NIPT SAMPLE COLLECTION PROCESS

REQUIRED INFORMATION

Patient Information

- Patient full name or unique identifier (Mandatory)
- Date of birth MM/DD/YYYY (Mandatory)
- Patient full address and phone number

Ordering Physician/Additional Reporting Information

- Doctor or ordering physician (Mandatory)
- Account name and Account number
- Full address, phone, and fax

Test Indication/Clinical Information

- Check test indication
- Fill in key clinical information
 - Patient's gestational age, weeks and days on MM/DD/YYYY (Mandatory)
 - Dating method (check a box)
 - Patient's blood draw date, MM/DD/YYYY (Mandatory)
 - Maternal Height (cm or ft)
 - Maternal Weight (kgs or lbs)
- Signatures
 - Maternal Height (cm or ft)
 - Maternal Height (cm or ft)

SAMPLE COLLECTION AND LABELING

Blood Draw Process

- Confirm all supplies are available – Streck tube, box, the requisition form, shipping materials, and FedEx bag
- Draw 7-10 mL of blood, and mix by gentle inversion 8-10 times. A minimum of 7 mL is required.

- Label the tube with BOTH the patient's full name or unique identifier, and date of birth (MM/DD/YYYY)
NOTE: Improper labeling of tube or inadequate sample volume will cause sample rejection

SHIPPING

- Place Streck tube into biohazard bag with absorbent sheet, along with completed TRF, and place sample bag into shipping container

- Send e-mail to dntest@premiergx.com with office location and hours of operation
- Request pick up the same day of draw